

Clinical evaluation report

Product name: SARS-CoV-2 Antigen Rapid Test Kit (Colloidal gold Immunoassay)

Wuhan UNscience Biotechnology Co., Ltd.

CONTENTS

SUMMARY	2
1 BASIC CONTENT	5
1.1 Introduction	5
1.2 Research objective	6
1.3 Experimental management	6
1.3.1 Quality control in clinical studies	6
1.3.1.1 Experimental control	6
1.3.1.2 Quality control of clinical case diagnosis	7
1.3.1.3 Laboratory quality control	7
1.3.2 Biosafety	7
1.3.3 Clinical data management and statistical analysis	7
1.4 Experimental design	8
1.4.1 The overall design and plan of the experiment	8
1.4.2 Test design and test method selection	8
1.4.2.1 Determination of the gold standard	8
1.4.2.2 Sample selection criteria and exclusion criteria	9
1.4.2.2.1 Sample type	9
1.4.2.2.2 Sample selection criteria	9
1.4.2.2.4 Sample rejection criteria	10
1.4.3 Collection, storage and transportation of samples	10
1.4.4 Information about products used in clinical trials	11
1.4.5 Methods of quality control for product testing	11
1.4.6 Result judgment	11
1.4.7 Statistical analysis	11
1.4.8 Modification of the scheme during the test	12
There was no modification of the scheme during the test.	12
1.5 Clinical trial results and analysis	12
1.5.1 Comparative test results	12
1.5.2 Coincidence rate statistics with reference system	28
1.5.3 Sensitivity and specificity	28
1.5.4 Results in homology	29
1.5.5 Homologous consistency	31
1.6 Discussion and Conclusions	31
2 A DESCRIPTION OF SPECIAL CIRCUMSTANCES IN CLINICAL STUDIES	32
3 WRITING TIME OF THE CLINICAL REPORT	32

SUMMARY

Product name: SARS-CoV-2 Antigen Rapid Test Kit (Colloidal gold Immunoassay).

Main researcher: Wuhan UNscience Biotechnology Co., Ltd.

Research purpose: The purpose of this test is to conduct clinical validation trials with SARS-CoV-2 antigen rapid test kit (Colloidal gold Immunoassay) produced by Wuhan UNscience Biotechnology Co., Ltd. according to Clinical investigation of medical devices for human subjects-Good clinical practice (ISO 14155:2011), Guidelines on medical devices Evaluation of clinical data: A guide for manufacturers and Notified Body, to evaluate whether the clinical application performance of the test reagent and the reference reagent on the market are equivalent.

Test description: The sample types in this experimental study are oropharyngeal swabs, nasal swabs and nasopharyngeal swabs. The test uses clinical comparison and data analysis with the gold standard to analyze whether the results are equivalent. Then use the test reagent to determine the oropharyngeal swab, nasal swab and nasopharyngeal swab samples to evaluate whether the test results of the oropharyngeal swab, nasal swab and nasopharyngeal swab are consistent.

From May 2020, 341 specimens of cases have been selected, including 109 specimens of patients with clinically confirmed cases of novel coronavirus infection and 232 specimens of clinically excluded cases. None of the patients were included that did not meet the protocol, and no cases of laboratory operation deviation were found. 341 cases were detected and the results were statistically analyzed to calculate the sensitivity and specificity of the methods; then use the test reagent to determine 50 oropharyngeal swab, nasal swab and nasopharyngeal swab samples from the same source to evaluate whether the test results are consistent, and obtain the following research results and conclusion:

1. Statistical analysis of 341 clinical samples (109 positive cases and 232 negative cases) showed the sensitivity is 96.330% (95%CI: 90.870%, 98.991%) and 99.569% (95%CI: 97.622%, 99.989%). The comparison results of 341 clinical samples with nucleic acid test shows that the positive coincidence rate is 96.33% and the negative

coincidence rate is 99.57%.

2. In addition, select homologous oropharyngeal swabs, nasal swabs and nasopharyngeal swabs from 50 subjects to compare, it shows that the consistent detection rate between oropharyngeal swabs, nasal swabs and nasopharyngeal swabs is 100% (95%CI: -100%, 100%).

The results indicate a high degree of consistency between this product and clinical diagnosis results.

ABBREVIATION

SARS-CoV-2 The novel coronavirus

1 Basic content

1.1 Introduction

The novel coronavirus (SARS-CoV-2) is a new type of coronavirus, which was named by the World Health Organization. It causes viral pneumonia with fever, fatigue, dry cough and sore throat as the main manifestations. In more severe cases, it can lead to pneumonia, severe acute respiratory syndrome, kidney failure, and even death. However, there is no specific treatment for it currently. In view of the epidemic trend of the virus in the world now, the rapid distinction between the infected and healthy people become the focus of epidemic prevention work.

The testing products on the market mainly include three categories: fluorescence quantitative PCR method, chemical/magnetic particle luminescence method and colloidal gold immunochromatographic method. Among them, the fluorescent quantitative PCR method is mainly used to measure the viral nucleic acid, while the chemical/magnetic particle luminescence method and the colloidal gold immunochromatography method are mainly for the antibodies detection in the infected cases.

The SARS-CoV-2 antigen rapid test kit (colloidal gold immunochromatography) produced by UNscience is used for in vitro qualitative detection of SARS-CoV-2 antigen in human oropharyngeal swabs, nasal swabs and nasopharyngeal swabs.

This kit adopts the sandwich method and the technical principle of colloidal gold immunochromatography to qualitative determine the SARS-CoV-2 antigen. During the test, the sample is dropped into the sample well, and chromatography is performed under the capillary effect. The SARS-CoV-2 antigen in the sample combined with the colloidal gold-labeled SARS-CoV-2 monoclonal antibody I, and then spread to the test area. It is captured by another coated antibody (SARS-CoV-2 monoclonal antibody II), to form a complex and gather in the test area (T line). The quality control area is coated with the goat anti-mouse antibody, and the colloidal gold-labeled antibody is captured to form a complex and aggregate in the quality control area (C line). If the C line does not show color, it indicates that the result is invalid, and this sample needs to be tested again.

1.2 Research objective

The purpose of this experiment is to take clinical test to SARS-CoV-2 Antigen Rapid Test Kit (Colloidal gold Immunoassay) produced by Wuhan UNscience Biotechnology Co., Ltd. in accordance with Clinical investigation of medical devices for human subjects - Good clinical practice (ISO 14155:2011), Guidelines on medical devices Evaluation of clinical data: A guide for manufacturers and Notified Body, to evaluate whether the test reagent is equivalent to the clinical application performance of the listed reference reagent.

1.3 Experimental management

1.3.1 Quality control in clinical studies

1.3.1.1 Experimental control

The person in charge of the project is responsible for this clinical evaluation test, and the test process shall strictly comply with the clinical evaluation project schemes.

The personnel who participate in the observation and collection of clinical data should have professional knowledge and skills and be fixed accordingly.

The researchers have a full understanding of each parameters in the clinical trial schemes after pre-clinical evaluation trial training. The required objective parameters should be inspected in accordance with the time and method specified in the schemes.

The researchers from Wuhan UNscience should be in charge of the whole operation of trial projected and record the data.

The researchers should verify the data that significantly deviates from the acceptable range, and make necessary explanations for it.

The researchers in charge of the clinical project check the progress of clinical trials regularly and verify the data and records carefully.

Supervision: During the trial process, the researchers in charge of the clinical project monitors and visits the clinical evaluation trial site regularly to ensure that the trial personnel conduct the experiment in accordance with the regulations and trial

evaluation schemes, and also to make sure the trial process records are complete, true and traceable.

1.3.1.2 Quality control of clinical case diagnosis

The Confirmed cases and excluded cases are determined by the Treatment of Novel Coronavirus Pneumonia (Trial Edition 7), issued by National Health Commission China, on March 3rd, 2020.

1.3.1.3 Laboratory quality control

The quality control of the Tuberculosis diagnostic laboratory is carried out according to the requirement of laboratory quality control regulated in the Clinical Technical Operation (fascicule Tuberculosis) compiled by Chinese Medical Association.

The quality control of the laboratory for testing products for clinical research shall comply with the specific requirements of basic laboratory operation specifications and product specifications, including basic temperature and humidity control, calibration of measuring instruments and equipment, etc. If the quality control conditions that do not meet the requirements of the product specification appear during the test, the re-inspection shall be required, and the qualified quality control data can be used only after passing the test.

Before and during the study, the applicant provided positive and negative reference materials to review and verify the specimen testing personnel and laboratory conditions.

1.3.2 Biosafety

Laboratory operations and specimen handling are conducted in accordance with the operating specifications of grade II (and above) bio-safety laboratories. The conductor is under grade III bio-safety protection.

1.3.3 Clinical data management and statistical analysis

The double-blind method was adopted in this clinical study. The study specimens of the selected cases were indicated serial number by clinical examiner, and sent to laboratory for testing. The case information and test data during the study were kept

separately by clinicians and inspectors. At the end of the study, case information and test data were collected in the department of the main investigator to uncover the blindness, and the results were judged and analyzed.

The person in charge of the laboratory appoints special test personnel and review personnel to be responsible for testing and recording the test data of each case.

1.4 Experimental design

1.4.1 The overall design and plan of the experiment

The selected samples in this trial refer to the “the Treatment of Novel Coronavirus Pneumonia (Trial Edition 7), issued by National Health Commission China, on March 3rd, 2020”, using the gold standard and the assessment result comparison method to comprehensively evaluate the clinical diagnostic performance of the products to be registered.

1.4.2 Test design and test method selection

1.4.2.1 Determination of the gold standard

The gold standard of new coronavirus diagnosis:

Suspected cases determined through epidemiological investigation and clinical symptoms, with pathogenic or serological evidence are confirmed cases.

1. Real-time fluorescent RT-PCR detection of new coronavirus nucleic acid positive;
2. Viral gene sequencing, highly homologous to the known new coronavirus;
3. New coronavirus-specific IgM antibodies and IgG antibodies are positive in serum;
4. New coronavirus-specific IgG antibody in serum changes from negative to positive or is 4 times or higher than the acute period in recovery period.

The gold standard for diagnosis of excluded cases: suspected cases determined by epidemic virus investigation and clinical symptoms, two consecutive new coronavirus nucleic acid tests are negative (at least 24 hours interval), and the specificity of new coronavirus is still negative 7 days after the onset of disease, that is exclude cases.

1.4.2.2 Sample selection criteria and exclusion criteria

The selected samples all refer to the requirements of the diagnosis and treatment specifications for confirmed and excluded cases, and the hospital will issue diagnosis and exclusion opinions on the cases.

1.4.2.2.1 Sample type

Normally collected oropharyngeal swab, nasal swab and nasopharyngeal swab samples.

1.4.2.2.2 Sample selection criteria

- ① The samples are the remaining samples after the routine testing of the laboratory department, which comes from designated hospitals;
- ② All samples are collected from clinically positive (confirmed) and negative (excluded) cases;
- ③ Interfering samples mainly choose samples with *Staphylococcus aureus*, *Streptococcus pneumoniae*, Measles virus, Mumps virus, Adenovirus type 3, *Mycoplasma pneumoniae*, Parainfluenza virus type 2, Metapneumovirus, Coronavirus OC43, Coronavirus 229E, *Bacillus parapertussis*, Type B influenza virus Victoria line, Type B influenza virus Y line, Type A influenza virus H1N1, Type A influenza virus H3N2, Avian influenza virus H7N9, Avian influenza H5N1 virus, Epstein barr virus, Enterovirus CA16 and Rhinovirus; Samples of patients treated with drugs such as Histamine dihydrochloride, Zanamivir, Ribavirin, IFN- α , Oseltamivir, Paramivir, Lopinavir, Ritonavir, Arbidol, Levofloxacin, Azithromycin, Ceftriaxone, Meropenem and Tobramycin, etc.

1.4.2.2.3 Sample exclusion criteria

- ① The sample volume is not enough to complete all the tests;
- ② Samples that have been contaminated, damaged or potentially contaminated or damaged;
- ③ Samples with incomplete patient information (such as sample type, department, preliminary diagnosis conclusion, etc.);

-
- ④ Samples that have not been collected, stored or stored for too long as required;
 - ⑤ Samples with duplicate patient information (same patient).

1.4.2.2.4 Sample rejection criteria

- ① Samples whose test results cannot be obtained due to reagents or human factors;

1.4.3 Collection, storage and transportation of samples

SAMPLE REQUIREMENTS:

① Oropharyngeal swab: The head of the person is slightly tilted, with mouth wide open, exposing the pharyngeal tonsils on both sides. Use the swab to gently wipe the tonsils on both sides for at least 3 times, and then wipe the posterior pharyngeal wall up and down at least 3 times.

② Nasal swab: Prior to collecting the nasal swab, the patient should be instructed to blow their nose. Carefully insert the swab into the nostril with the most secretion under visual inspection. Using gentle rotation, push the swab until resistance is met at the level of the turbinates (less than one inch into the nostril), and rotate the swab against the nasal wall several times and then remove it from the nostril.

③ Nasopharyngeal swab: Carefully insert the swab into the nostril with the most secretion under visual inspection. Keep the swab near the septum floor of the nose while gently pushing the swab into the posterior nasopharynx. Rotate the swab several times then remove it from the nasopharynx (in case of reflex cough, stop for 1 minute).

SAMPLE PREPARATION:

Take out sampling tube and add 10 drops of sample treatment solution into it. Put the swab into sampling tube, make sure the swab soaked in the solution. Rotate and squeeze the swab on the wall and bottom of the tube 10 times, squeeze the swab tip along the inner wall of the sample tube to keep as much solution in the tube as possible. Remove the swab and test. It is recommended to test immediately after sample collection and processing. If the test cannot be performed timely, the processed samples can be stored at 2-8°C for 48h.

All samples were heat-inactivated at 56 °C for 30 minutes after collection.

1.4.4 Information about products used in clinical trials

Research reagents

Product Name: SARS-CoV-2 Antigen Rapid Test Kit (Colloidal gold Immunoassay)

Size: 25T/kit

Manufacturer: Wuhan UNscience Co. Ltd.

Lot. No.: 20200509

Reference reagents

The COVID-19 RT-PCR test kits used by the reference hospitals on the samples test are mainly Novel Coronavirus 2019-nCoV nucleic acid detection kit (fluorescence PCR method) from BGI BIOTECHNOLOGY (WUHAN) CO., LTD. This product has registered in NMPA and got the approval No. 20203400060.

1.4.5 Methods of quality control for product testing

The minimum detection range of reference product of S1 should be negative, S3 should be positive, and S2 can be negative or positive.

1.4.6 Result judgment

Observe and record the results after 10 minutes.

Positive: Both the test line (T line) and the quality control line (C line) appear colors.

Negative: The test line (T line) appears no color and the quality control line (C line) appears color.

Invalid: The quality control line (C line) does not appear color, indicating that this test is invalid, it should be tested again.

1.4.7 Statistical analysis

Statistical analysis was performed using MEDCALC software. Percentage of diagnostic performance indicators such as sensitivity, specificity, positive consistency, negative consistency and overall consistency were reported and their 95% confidence intervals were calculated.

1.4.8 Modification of the scheme during the test

There was no modification of the scheme during the test.

1.5 Clinical trial results and analysis

1.5.1 Comparative test results

341 cases were included, including 109 cases of novel Coronavirus infection with clinically confirmed patients and 232 cases of clinical excluded cases. No cases were included in accordance with the protocol and no cases of laboratory operation deviation were found. 341 cases were detected and the results were statistically analyzed to calculate the sensitivity and specificity of the methods.

Table 1 Clinical trial results

No.	Age	Days onset	Clinical diagnosis	Nucleic acid	Sample type	Ct value	SARS-CoV-2 Ag	Sample type
1	38	6	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
2	77	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
3	38	7	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
4	49	5	+	+	Oropharyngeal swab	31	+	Nasopharyngeal swab
5	43	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
6	52	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
7	47	6	+	+	Nasopharyngeal swab	33	+	Oropharyngeal swab
8	51	2	+	+	Nasopharyngeal swab	27	+	Oropharyngeal swab
9	81	6	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
10	25	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
11	78	7	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

12	57	7	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
13	78	6	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
14	48	3	+	+	Nasopharyngeal swab	29	+	Oropharyngeal swab
15	50	6	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
16	30	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
17	10	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
18	56	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
19	48	6	+	+	Oropharyngeal swab	28	+	Nasopharyngeal swab
20	62	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
21	82	7	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
22	11	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
23	79	3	+	+	Nasopharyngeal swab	29	+	Oropharyngeal swab
24	45	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
25	31	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
26	13	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
27	62	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
28	0	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
29	22	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
30	72	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
31	54	7	+	+	Nasopharyngeal swab	25	+	Oropharyngeal swab
32	41	7	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
33	32	7	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab

34	74	6	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
35	78	6	+	+	Nasopharyngeal swab	33	+	Oropharyngeal swab
36	45	2	+	+	Nasopharyngeal swab	30	-	Oropharyngeal swab
37	23	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
38	55	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
39	82	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
40	10	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
41	76	7	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
42	5	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
43	64	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
44	50	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
45	83	6	+	+	Oropharyngeal swab	30	+	Nasopharyngeal swab
46	39	7	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
47	69	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
48	60	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
49	9	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
50	83	7	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
51	81	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
52	24	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
53	26	6	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
54	53	7	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
55	61	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab

56	69	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
57	72	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
58	58	7	+	+	Oropharyngeal swab	27	+	Nasopharyngeal swab
59	47	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
60	24	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
61	35	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
62	80	6	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab
63	17	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
64	81	3	+	+	Nasopharyngeal swab	32	+	Oropharyngeal swab
65	64	7	+	+	Nasopharyngeal swab	24	+	Oropharyngeal swab
66	37	1	+	+	Nasopharyngeal swab	33	+	Oropharyngeal swab
67	22	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
68	2	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
69	61	4	+	+	Nasopharyngeal swab	33	+	Oropharyngeal swab
70	79	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
71	73	1	+	+	Oropharyngeal swab	32	-	Nasopharyngeal swab
72	55	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
73	69	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
74	41	4	+	+	Oropharyngeal swab	35	-	Nasopharyngeal swab
75	28	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
76	26	6	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
77	77	6	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

78	44	7	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
79	75	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
80	50	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
81	30	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
82	31	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
83	15	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
84	56	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
85	67	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
86	27	1	-	-	Oropharyngeal swab	NEG	+	Nasopharyngeal swab
87	59	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
88	65	6	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
89	30	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
90	58	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
91	62	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
92	31	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
93	83	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
94	68	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
95	24	5	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab
96	52	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
97	53	6	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
98	82	4	+	+	Nasopharyngeal swab	24	+	Oropharyngeal swab
99	31	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

100	56	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
101	52	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
102	38	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
103	10	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
104	75	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
105	29	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
106	46	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
107	55	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
108	30	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
109	79	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
110	19	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
111	32	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
112	63	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
113	46	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
114	38	1	+	+	Nasopharyngeal swab	23	+	Oropharyngeal swab
115	35	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
116	24	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
117	58	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
118	64	7	+	+	Nasopharyngeal swab	30	+	Oropharyngeal swab
119	75	6	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
120	39	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
121	65	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab

122	46	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
123	32	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
124	51	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
125	43	5	+	+	Nasopharyngeal swab	27	+	Oropharyngeal swab
126	54	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
127	44	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
128	35	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
129	62	7	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
130	69	5	+	+	Nasopharyngeal swab	31	+	Oropharyngeal swab
131	62	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
132	57	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
133	28	4	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab
134	53	1	+	+	Nasopharyngeal swab	28	+	Oropharyngeal swab
135	58	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
136	71	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
137	35	7	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
138	60	1	+	+	Oropharyngeal swab	23	+	Nasopharyngeal swab
139	53	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
140	18	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
141	72	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
142	46	6	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
143	72	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

144	80	2	+	+	Nasopharyngeal swab	30	+	Oropharyngeal swab
145	53	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
146	45	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
147	9	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
148	59	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
149	35	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
150	60	7	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
151	40	7	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
152	60	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
153	75	7	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
154	60	7	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
155	71	6	+	+	Nasopharyngeal swab	34	+	Oropharyngeal swab
156	62	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
157	75	7	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
158	74	6	+	+	Oropharyngeal swab	31	+	Nasopharyngeal swab
159	8	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
160	10	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
161	26	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
162	52	4	+	+	Nasopharyngeal swab	27	+	Oropharyngeal swab
163	56	5	+	+	Nasopharyngeal swab	25	+	Oropharyngeal swab
164	67	6	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab
165	51	5	+	+	Nasopharyngeal swab	25	+	Oropharyngeal swab

166	18	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
167	21	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
168	21	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
169	65	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
170	27	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
171	18	5	+	+	Oropharyngeal swab	25	+	Nasopharyngeal swab
172	58	4	+	+	Oropharyngeal swab	27	+	Nasopharyngeal swab
173	52	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
174	34	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
175	48	4	+	+	Nasopharyngeal swab	23	+	Oropharyngeal swab
176	44	5	+	+	Oropharyngeal swab	28	+	Nasopharyngeal swab
177	36	4	+	+	Oropharyngeal swab	26	+	Nasopharyngeal swab
178	70	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
179	70	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
180	51	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
181	31	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
182	48	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
183	63	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
184	15	5	+	+	Nasopharyngeal swab	30	+	Oropharyngeal swab
185	31	7	+	+	Oropharyngeal swab	29	+	Nasopharyngeal swab
186	32	3	+	+	Nasopharyngeal swab	23	+	Oropharyngeal swab
187	36	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

188	49	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
189	36	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
190	68	7	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab
191	33	5	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab
192	7	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
193	70	3	+	+	Oropharyngeal swab	31	+	Nasopharyngeal swab
194	44	6	+	+	Nasopharyngeal swab	27	+	Oropharyngeal swab
195	45	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
196	3	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
197	26	5	+	+	Oropharyngeal swab	26	+	Nasopharyngeal swab
198	20	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
199	38	4	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab
200	5	4	+	+	Oropharyngeal swab	29	+	Nasopharyngeal swab
201	33	7	+	+	Nasopharyngeal swab	31	+	Oropharyngeal swab
202	68	4	+	+	Nasopharyngeal swab	31	+	Oropharyngeal swab
203	34	4	+	+	Oropharyngeal swab	33	+	Nasopharyngeal swab
204	36	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
205	17	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
206	14	4	+	+	Nasopharyngeal swab	23	+	Oropharyngeal swab
207	2	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
208	36	7	+	+	Oropharyngeal swab	28	+	Nasopharyngeal swab
209	29	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

210	13	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
211	38	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
212	67	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
213	7	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
214	4	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
215	11	6	+	+	Nasopharyngeal swab	32	+	Oropharyngeal swab
216	66	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
217	21	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
218	1	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
219	30	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
220	61	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
221	30	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
222	44	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
223	63	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
224	59	7	+	+	Nasopharyngeal swab	30	+	Oropharyngeal swab
225	3	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
226	32	2	+	+	Nasopharyngeal swab	30	+	Oropharyngeal swab
227	28	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
228	39	5	+	+	Nasopharyngeal swab	25	+	Oropharyngeal swab
229	47	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
230	39	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
231	30	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

232	53	4	+	+	Nasopharyngeal swab	34	-	Oropharyngeal swab
233	2	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
234	25	7	+	+	Oropharyngeal swab	35	+	Nasopharyngeal swab
235	32	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
236	4	6	+	+	Oropharyngeal swab	26	+	Nasopharyngeal swab
237	4	4	+	+	Nasopharyngeal swab	31	+	Oropharyngeal swab
238	41	6	+	+	Nasopharyngeal swab	31	+	Oropharyngeal swab
239	18	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
240	56	6	+	+	Oropharyngeal swab	33	+	Nasopharyngeal swab
241	33	5	+	+	Oropharyngeal swab	31	+	Nasopharyngeal swab
242	17	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
243	40	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
244	63	6	+	+	Oropharyngeal swab	25	+	Nasopharyngeal swab
245	61	5	+	+	Oropharyngeal swab	27	+	Nasopharyngeal swab
246	15	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
247	23	5	+	+	Nasopharyngeal swab	30	+	Oropharyngeal swab
248	58	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
249	47	4	+	+	Oropharyngeal swab	31	+	Nasopharyngeal swab
250	58	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
251	36	5	+	+	Oropharyngeal swab	26	+	Nasopharyngeal swab
252	61	5	+	+	Nasopharyngeal swab	27	+	Oropharyngeal swab
253	13	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

254	5	3	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
255	25	2	+	+	Oropharyngeal swab	26	+	Nasopharyngeal swab
256	44	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
257	6	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
258	7	5	+	+	Oropharyngeal swab	30	+	Nasopharyngeal swab
259	47	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
260	15	5	+	+	Oropharyngeal swab	30	+	Nasopharyngeal swab
261	14	3	+	+	Nasopharyngeal swab	28	+	Oropharyngeal swab
262	51	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
263	53	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
264	21	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
265	25	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
266	64	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
267	39	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
268	32	3	+	+	Oropharyngeal swab	26	+	Nasopharyngeal swab
269	13	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
270	60	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
271	28	3	+	+	Nasopharyngeal swab	26	+	Oropharyngeal swab
272	65	7	+	+	Nasopharyngeal swab	24	+	Oropharyngeal swab
273	16	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
274	3	7	+	+	Oropharyngeal swab	24	+	Nasopharyngeal swab
275	38	7	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab

276	41	3	+	+	Oropharyngeal swab	30	+	Nasopharyngeal swab
277	27	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
278	46	7	+	+	Oropharyngeal swab	31	+	Nasopharyngeal swab
279	25	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
280	9	4	+	+	Nasopharyngeal swab	27	+	Oropharyngeal swab
281	39	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
282	8	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
283	32	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
284	5	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
285	36	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
286	47	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
287	30	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
288	10	7	+	+	Oropharyngeal swab	29	+	Nasopharyngeal swab
289	24	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
290	57	4	+	+	Nasopharyngeal swab	26	+	Oropharyngeal swab
291	30	5	+	+	Nasopharyngeal swab	26	+	Oropharyngeal swab
292	68	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
293	36	6	+	+	Oropharyngeal swab	26	+	Nasopharyngeal swab
294	64	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
295	12	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
296	64	4	+	+	Oropharyngeal swab	25	+	Nasopharyngeal swab
297	31	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

298	1	1	+	+	Oropharyngeal swab	26	+	Nasopharyngeal swab
299	33	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
300	31	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
301	57	6	+	+	Oropharyngeal swab	23	+	Nasopharyngeal swab
302	40	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
303	32	5	+	+	Oropharyngeal swab	27	+	Nasopharyngeal swab
304	63	3	+	+	Nasopharyngeal swab	25	+	Oropharyngeal swab
305	54	4	+	+	Nasopharyngeal swab	28	+	Oropharyngeal swab
306	10	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
307	14	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
308	18	6	+	+	Nasopharyngeal swab	32	+	Oropharyngeal swab
309	30	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
310	6	1	+	+	Nasopharyngeal swab	23	+	Oropharyngeal swab
311	40	4	+	+	Oropharyngeal swab	30	+	Nasopharyngeal swab
312	46	1	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
313	3	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
314	45	5	+	+	Nasopharyngeal swab	30	+	Oropharyngeal swab
315	69	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
316	23	5	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab
317	22	1	+	+	Nasopharyngeal swab	31	+	Oropharyngeal swab
318	1	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
319	25	2	+	+	Oropharyngeal swab	32	+	Nasopharyngeal swab

320	2	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
321	66	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
322	23	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
323	27	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
324	43	3	+	+	Oropharyngeal swab	27	+	Nasopharyngeal swab
325	59	4	+	+	Oropharyngeal swab	29	+	Nasopharyngeal swab
326	67	7	+	+	Oropharyngeal swab	28	+	Nasopharyngeal swab
327	16	6	+	+	Nasopharyngeal swab	25	+	Oropharyngeal swab
328	28	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
329	42	5	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
330	10	7	+	+	Nasopharyngeal swab	28	+	Oropharyngeal swab
331	7	1	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
332	33	3	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
333	17	7	+	+	Nasopharyngeal swab	25	+	Oropharyngeal swab
334	10	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
335	23	6	+	+	Oropharyngeal swab	29	+	Nasopharyngeal swab
336	63	5	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
337	20	4	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
338	49	2	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab
339	30	4	+	+	Oropharyngeal swab	30	+	Nasopharyngeal swab
340	43	2	-	-	Nasopharyngeal swab	NEG	-	Oropharyngeal swab
341	3	4	-	-	Oropharyngeal swab	NEG	-	Nasopharyngeal swab

1.5.2 Coincidence rate statistics with reference system

Statistically analyze on 341 samples, the statistical results of the coincidence rate with the reference system are as follows:

Table 2. Coincidence rate with the reference system

Compared with nucleic acid test results			
	Pos by BGI	Neg by BGI	Total
Pos by UNScience	105	1	106
Neg by UNScience	4	231	235
Total	109	232	341
Positive coincidence rate	96.33%		
Negative coincidence rate	99.57%		

1.5.3 Sensitivity and specificity

Make statistics on 341 samples by software “Medcalc”, which includes 109 clinically confirmed cases and 232 excluded cases. There are 106 positive results, and 235 negative results with rapid kit. The clinical sensitivity and specificity of this product is 96.330% (95%CI: 90.870%, 98.991%) and 99.569% (95%CI: 97.622%, 99.989%).

Table 3. Product sensitivity and specificity

Sensitivity	96.330%	90.870% to 98.991%
Specificity	99.569%	97.622% to 99.989%
AUC	0.979	0.958 to 0.992
Positive Likelihood Ratio	223.486	31.602 to 1580.449
Negative Likelihood Ratio	0.037	0.014 to 0.096
Disease prevalence	31.965%	27.044% to 37.202%
Positive Predictive Value	99.057%	93.690% to 99.866%
Negative Predictive Value	98.298%	95.665% to 99.343%

1.5.4 Conclusion on Ct value, age, day onset

106 positive samples ct values were analyzed, coincidence rates as follows:

Range of Ct value	POS by PCR	POS by SARS-CoV-2 Ag	Coincidence rate
23-25	21	21	100%
26-29	37	37	100%
30-33	47	45	95.74%
34-35	4	2	50%

341 samples were analyzed, POSTIVE coincidence rates by age as follows:

Age	Amount	Pos by clinical diagnosis	POS by PCR	POS by SARS-CoV-2 Ag
0-20	67	19	19	19/19 (100%)
21-60	195	65	65	63/65 (96. 92%)
61-90	79	25	25	24/25 (96%)

341 samples were analyzed, POSTIVE coincidence rates by symptom onset days as follows:

Days onset	Amount	Pos by clinical diagnosis	POS by PCR	POS by SARS-CoV-2 Ag
1	42	8	8	7/8 (87. 5%)
2	51	6	6	5/6 (83. 33)
3	57	11	11	11/11 (100%)
4	64	23	23	21/23 (91. 3%)
5	60	23	23	23/23 (100%)
6	31	18	18	18/18 (100%)
7	36	20	20	20/20 (100%)

1.5.4 Results in homology

Apply the examination reagent to determine 50 samples of the homologous oropharyngeal swabs, nasal swabs and nasopharyngeal swabs, and evaluate whether the detection results of different sampling sites are consistent. The results are as follows:

Table 4. Results in homology

No.	Age	Days of symptoms onset	Result	Sampling type	Result	Sampling type	Result	Sampling type
1	51	4	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
2	50	7	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
3	52	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
4	46	6	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
5	65	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
6	55	7	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
7	59	5	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
8	64	6	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab

9	46	6	+	Oropharyngeal swab	+	Nasopharyngeal swab	+	Nasal swab
10	59	2	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
11	45	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
12	42	2	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
13	59	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
14	45	2	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
15	62	7	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
16	57	1	+	Oropharyngeal swab	+	Nasopharyngeal swab	+	Nasal swab
17	72	4	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
18	74	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
19	31	6	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
20	67	6	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
21	68	7	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
22	41	5	+	Oropharyngeal swab	+	Nasopharyngeal swab	+	Nasal swab
23	57	6	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
24	63	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
25	40	7	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
26	56	4	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
27	53	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
28	71	4	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
29	74	1	+	Oropharyngeal swab	+	Nasopharyngeal swab	+	Nasal swab
30	49	1	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
31	62	4	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
32	30	6	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
33	42	7	+	Oropharyngeal swab	+	Nasopharyngeal swab	+	Nasal swab
34	70	7	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
35	30	6	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
36	43	2	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
37	70	1	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
38	59	1	+	Oropharyngeal swab	+	Nasopharyngeal swab	+	Nasal swab
39	75	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
40	52	2	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
41	60	7	+	Oropharyngeal swab	+	Nasopharyngeal swab	+	Nasal swab
42	41	6	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
43	68	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
44	47	4	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
45	43	7	+	Oropharyngeal swab	+	Nasopharyngeal swab	+	Nasal swab
46	61	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
47	39	4	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
48	67	7	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
49	63	3	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab
50	35	2	-	Oropharyngeal swab	-	Nasopharyngeal swab	-	Nasal swab

1.5.5 Homologous consistency

Analyze on detection results of 50 samples of oropharyngeal swabs, nasal swabs and nasopharyngeal swabs and the results are as follows:

Homologous consistency of oropharyngeal swabs, nasal swabs and nasopharyngeal swabs:

Variable Y	Oropharyngeal swab
Variable X	Nasopharyngeal swab
Variable Z	Nasal swab

The sampling size	50
Consistency correlation coefficient	1.0000
95% confidence interval	-1.0000 - -1.0000
Pearson ρ (precise)	1.0000
Deviation correction factor C_b (accuracy)	1.0000

1.6 Discussion and Conclusions

Select 341 cases including 109 cases of clinically confirmed patients of novel Coronavirus infection and 232 cases of clinical excluded patients. No cases selected that are not in accordance with the protocol, no cases of laboratory operation deviation are found. Detect in 341 cases, statistically analyze the results and calculate the sensitivity and specificity. Then determine the 50 samples of homologous oropharyngeal swabs, nasal swabs and nasopharyngeal swabs with assessment reagent to evaluate whether the detection results are consistent. The following results and conclusions are obtained:

1. Statistically analyze on 109 clinically confirmed cases and 232 excluded cases. There are 106 positive results, and 235 negative results. The clinical sensitivity and specificity of this product is 96.330% (95%CI: 90.870%, 98.991%) and 99.569% (95%CI: 97.622%, 99.989%).
2. The test results of homologous oropharyngeal swabs, nasal swabs and nasopharyngeal swabs from 50 subjects are compared. The results show that the consistent detection rate between oropharyngeal swabs, nasal swabs and

nasopharyngeal swabs is 100% (95%CI: -100%, 100%).

The results show that the product is highly consistent with the clinical diagnosis.

2 A description of special circumstances in clinical studies

None.

3 Writing time of the Clinical report

The clinical report was completed on November 16, 2020.